Keratoplasty

Kératoplastie

Queratoplastia

SINCE 1820
200 YEARS
MORIA • OPHTHALMIC INSTRUMENTS

Moria
Ophthalmic Instruments

For 200 years, Moria has been known for its expertise in making ophthalmic microsurgical instruments. All our instruments are designed, manufactured and hand finished in France. Moria product ranges are covering the following area of expertise:

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**All surgeries
Steel reusable
instruments**

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**All surgeries
Single Use
instruments**

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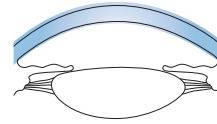
One Use

Système rotatif manuel
Sistema rotativo manual

Accuracy and reproducibility for DSAEK graft preparation

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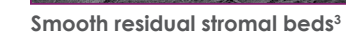
MICROKERATOMES

Microkératomes/ Microqueratomos

One Use-Plus for Artificial Chamber
Automated, linear, compatible with low temperature sterilization using hydrogen peroxide ("plasma sterilization").

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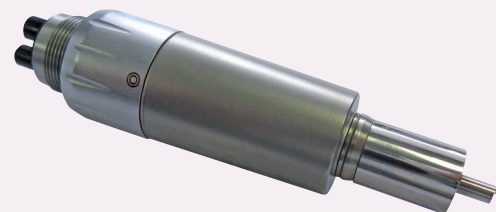
One Use-Plus
Automated DSAEK
DSA EK

19175*

One Use Turbine
Manual, linear, sterilization by autoclave.

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One Use
DSA EK

19155*

CBm Turbine
Manual, linear, sterilization by autoclave.

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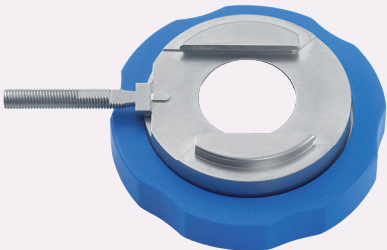
MICROKERATOME ACCESSORIES

Accessoires Microkératomes / Accesorios Microqueratomo

Single-Use Guide-Ring for One Use-Plus handpiece (#19175)

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
19187

Reusable Guide-Ring for One Use-Plus handpiece (#19175)

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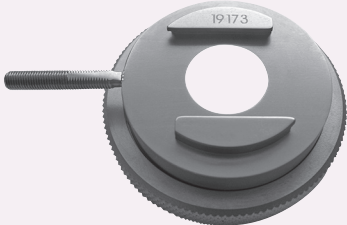
DSA EK

19173

Single-Use Guide-Ring for One Use turbine (#19155)

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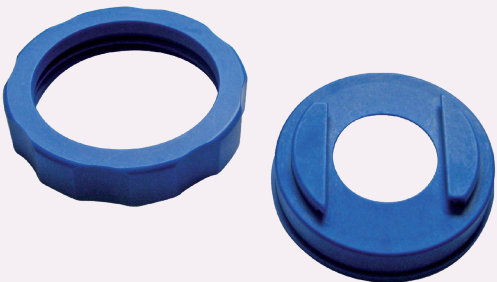
19186

Reusable Guide-Ring for One Use turbine (#19155)

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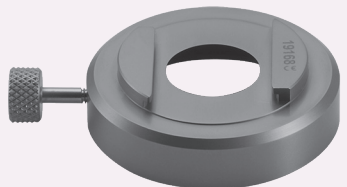MICROKERATOME ACCESSORIES

Accessoires Microkératomes / Accesorios Microqueratomo

Single-Use Guide-Ring for CBm turbine (#19303)

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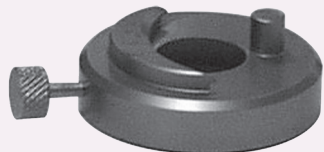
DSA EK

19180

Reusable Guide-Ring for CBm turbine (#19303)

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ALTK DSA EK

19171

CBm-ALTK suction ring for recipient Available in 4 sizes: -1, 0, +1, +2 Please consult us.

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19182

Single-Use Artificial Chamber for Single-Use Trephines

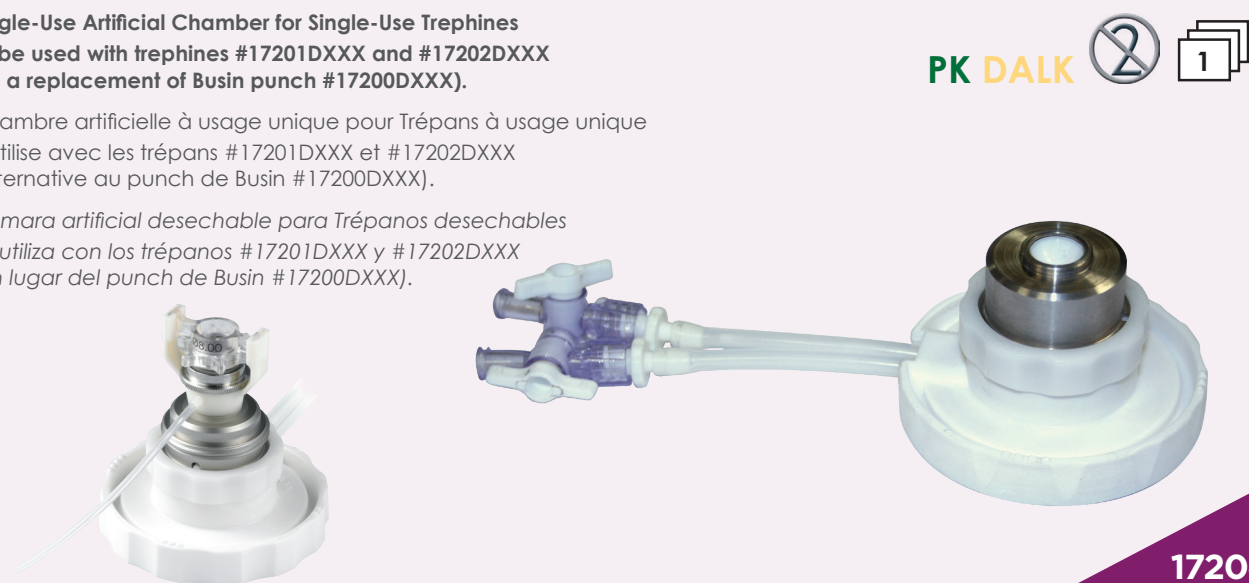
To be used with trephines #17201DXXX and #17202DXXX (as a replacement of Busin punch #17200DXXX).

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Chambres Artificielles / Cámaras Artificiales

Reusable Artificial Chamber

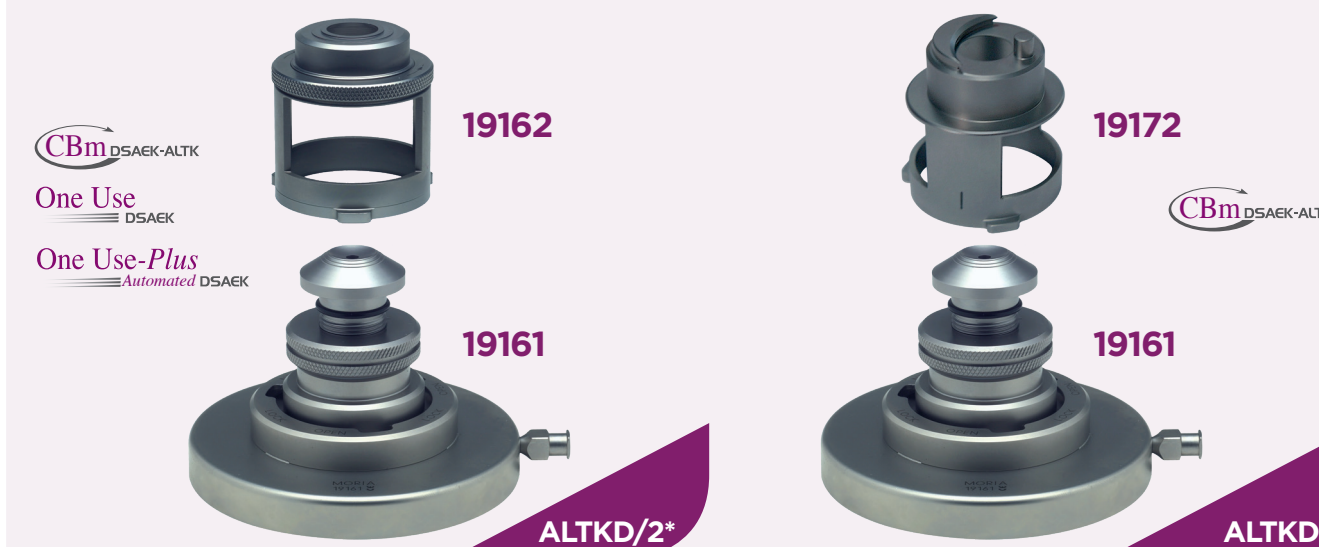
To be used with One Use-Plus, One Use and CBm microkeratomes (#19175, #19155 and #19303).

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
Single-use Tubing Set for Artificial Chamber

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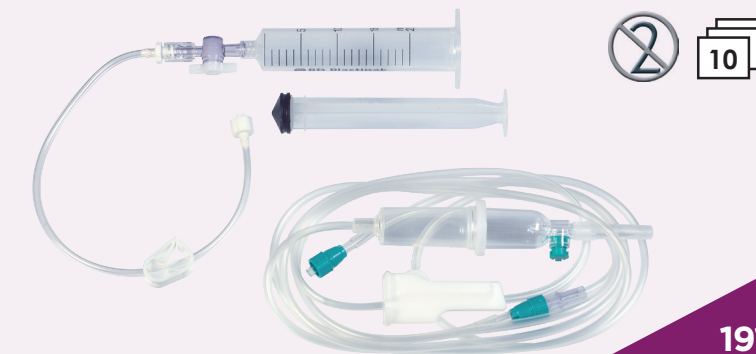Punchs Donneur / Puncchs Donante

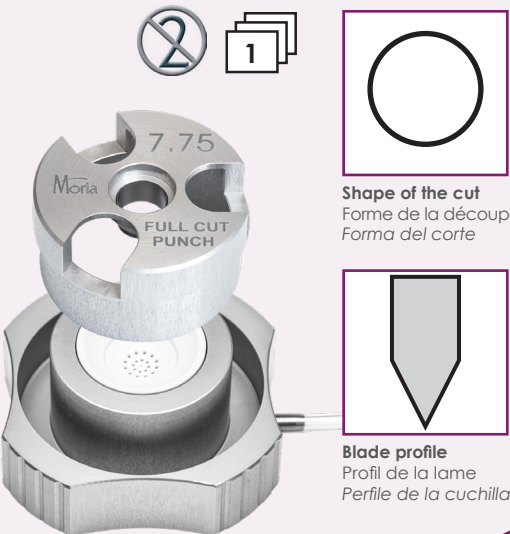
PK-DALK-DSA EK

Deep Well Full-Cut Punch
With aspiration
Straight-walled cut (symmetrical double-bevel blade) throughout the thickness of the graft
Anatomical Deep Well with double curvature to envelop the cornea in its most natural position

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
17214DXXX

Busin Punch
With aspiration
Straight-walled cut (symmetrical double-bevel blade) throughout the thickness of the graft

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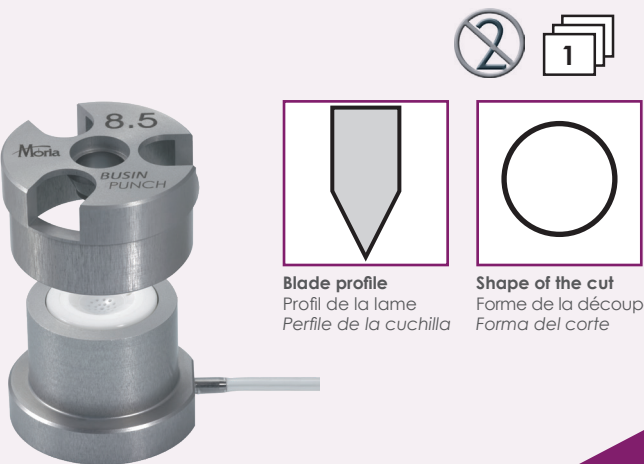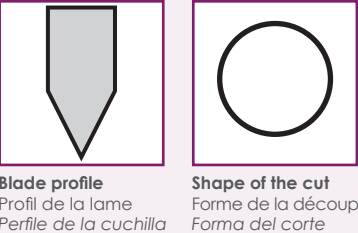
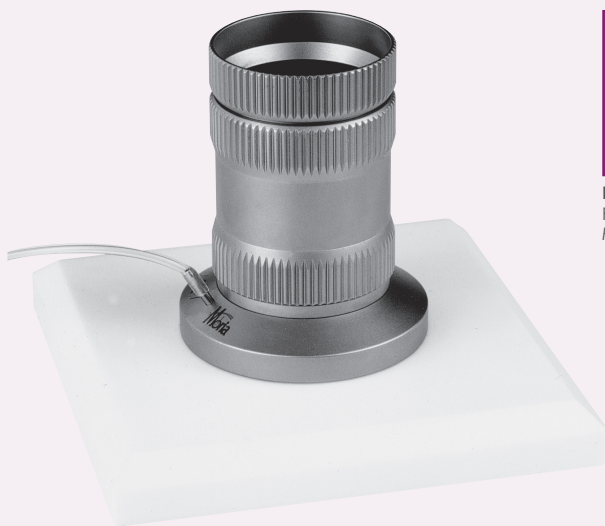
17171P

Two-colored Teflon®/Propylux® Receptacle Block for Donor Punch #17169

Réceptacle bi-couleur en Teflon®/Propylux® pour Punch Donneur #17169
Pastilla con dos colores de Teflon®/Propylux® para Punch Donante #17169

17171PB

DONOR PUNCHES

Punchs Donneur / Puncchs Donante

► DMEK

Deep Well Guarded Punch
With aspiration
Straight-walled cut (symmetrical double-bevel blade)
Guarded blade
Anatomical Deep Well with double curvature to envelop the cornea in its most natural position.

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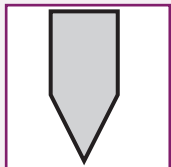
Deep Well Guarded Cover
Straight-walled cut (symmetrical double-bevel blade)
Guarded blade for peripheral scoring

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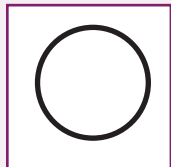
Punchs Donneur / Puncchs Donante

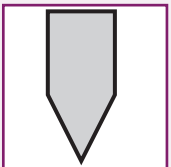
► DMEK

Guarded Punch
With aspiration
Straight-walled cut (symmetrical double-bevel blade)

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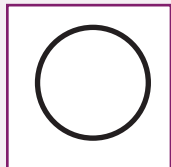
Shape of the incision
Forme de l'incision
Forma de la incisión

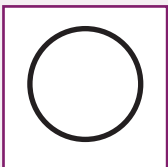
17207DXXX

XL Guarded Punch
Same features as Guarded Punch #17207Dxxx with a deeper guard

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Punchs Donneur / Punchs Donante

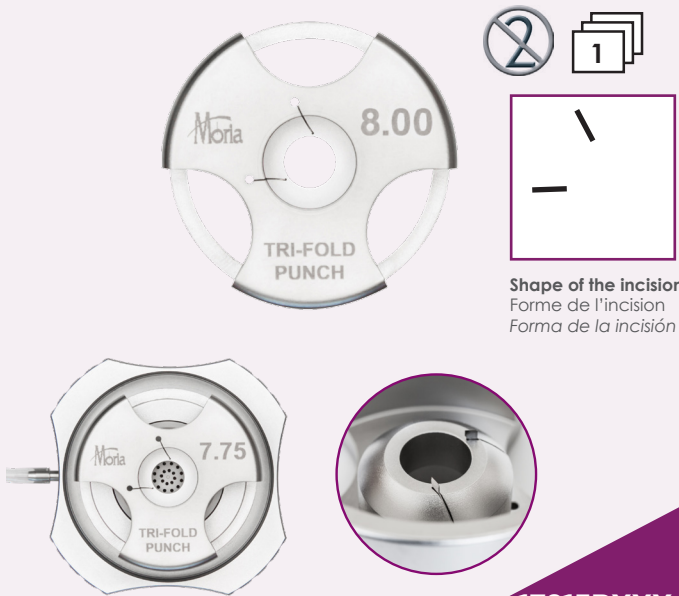
► DMEK

Deep Well Tri-Fold Cover
To be used in combination with a Deep Well Guarded Punch (#17212DXXX)(same diameter)
2 peripheral radial blades to induce a Tri-Scroll conformation that facilitates DMEK graft unscrolling and manipulation

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Available in / Disponible en / Disponible en :
7.50 mm, 7.75 mm, 8.00 mm


17215RXXX

Muraine Punch
With aspiration
Straight-walled cut (symmetrical double-bevel blade)
Obtention of 2 opposite hinges
Ideal for DMEK following Pr Muraine's technique

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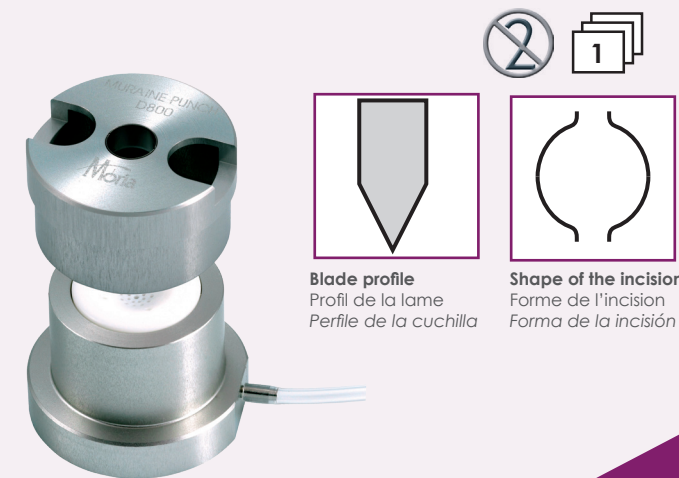Tréfans Receveur / Trépanos Receptor

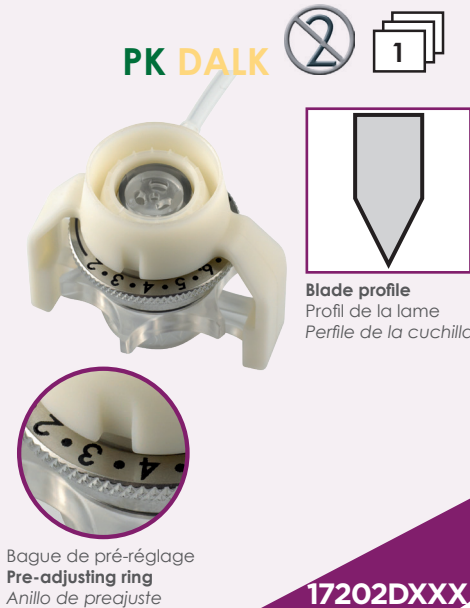
PK-DALK

Adjustable Trephine (for donor and recipient)
Pre-adjustment of the desired depth, symmetrical double-bevel blade.
Centering guide in contact with the cornea.
Each turn corresponds to a blade descent of 250 µm.
Can be used with artificial chamber #17204.

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Simple Trephine (for donor and recipient)
Straight-walled cut: symmetrical double-bevel blade.
Centering guide in contact with the cornea.
Each turn corresponds to a blade descent of 250 µm.
Can be used with artificial chamber #17204.

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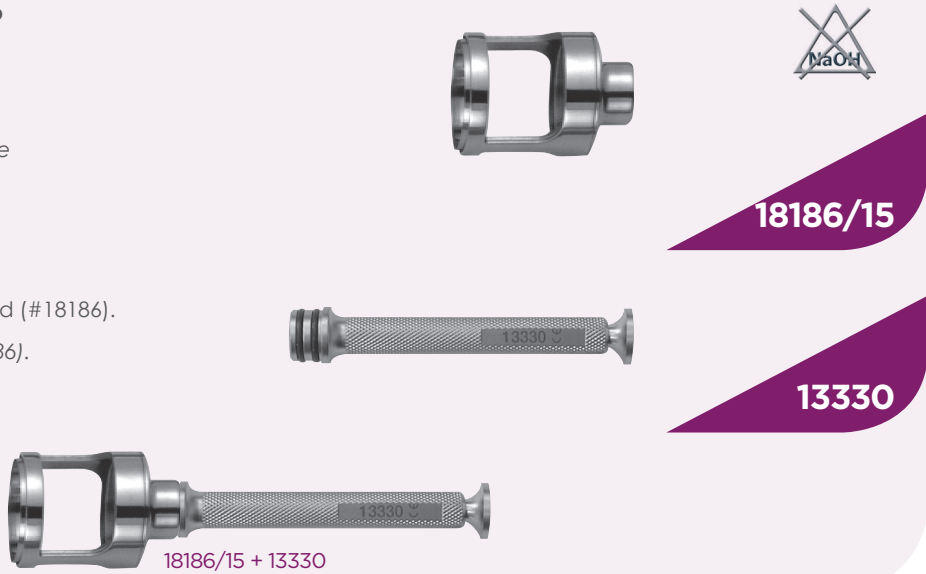
Tréfans Receveur / Trépanos Receptor

PK

Reusable Dezard trephine with limit stop
Diameter 15 mm.

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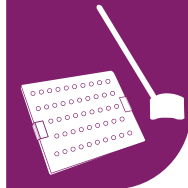
Reusable Silicone Block Receptacle Hollowed
Not sterile

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▶ ALTK

ALTK Applanating Lenses for Recipient

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19311

ALTK Applanating Lenses for Donor

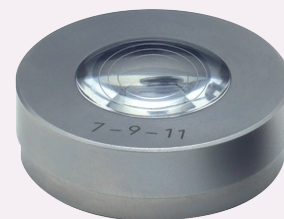
To be used with reusable artificial chamber
(#19161, #19162).

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
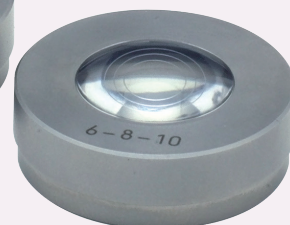
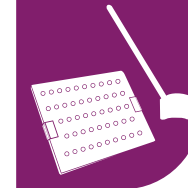
Sterilization box & tray for CBm and One Use turbines

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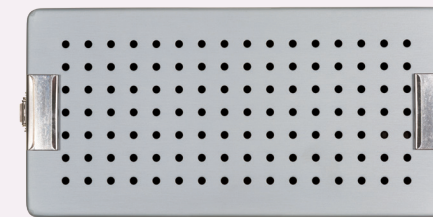
CBm
DSA EK-ALTK

22519139

Sterilisation tray for CBm heads & rings

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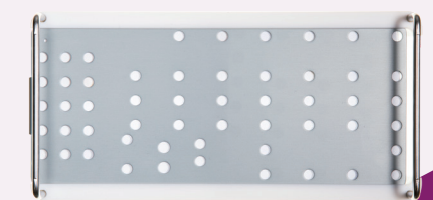CBm
DSA EK-ALTK

22519343

Sterilization box for reusable artificial chamber

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22519167

Storage box for ALTK applanating lenses

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22519169

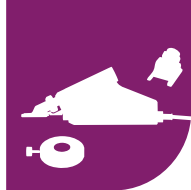
Storage box for One Use-Plus microkeratome for artificial chamber

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
GO

E0

Automated system	Manual system		
One Use-Plus <i>Automated</i> DSAEK	One Use DSAEK	CBm DSAEK-ALTK	
 TDM/4	 TD1/4	 TD4/4	 TD4/3
 ALTKD/2	 ALTKD/2	 ALTKD/2	 ALTKD

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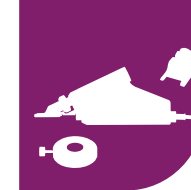
ALTKD	Reusable Artificial Chamber	Qty
19161	Reusable Artificial Chamber: Base	1
19172	Reusable Artificial Chamber: CBm Cover	1
22519167	Sterilization Box for Reusable Artificial Chamber	1
65009	Reusable Artificial Chamber User Manual	1

ALTKD/2	Reusable Artificial Chamber	Qty
19161	Reusable Artificial Chamber: Base	1
19162	Reusable Artificial Chamber: Cover	1
22519167	Sterilization Box for Reusable Artificial Chamber	1
65009	Reusable Artificial Chamber User Manual	1

TDM/4	OUP set for Reusable Artificial Chamber	Qty
19175	One Use-Plus Handpiece for Artificial Chamber	1
65106	Addendum of OUP Handpiece for Artificial Chamber	1
19173	Reusable Guide Ring for One Use-Plus Handpiece	1
22519174	Storage box for One Use-Plus Handpiece	1
19117	Moist Chamber	1
65101	One Use Head Selection (Nomogram)	1
65009	Reusable Artificial Chamber User Manual	1

TD1/4	One Use set for Reusable Artificial Chamber	Qty
19155	One Use Turbine	1
65007	One & One Use User manual	1
19353	Turbine Hose	1
19120	Nitrogen Gas Supply Hose	1
19168	Reusable Guide Ring for One Use Turbine	1
65104	Addendum of Reusable Artificial Chamber User Manual for One Use turbine	1
22519139	Sterilization Box & Tray for CBm and One Use Turbines	1
19117	Moist Chamber	1
65009	Reusable Artificial Chamber User Manual	1
65101	One Use Head Selection (Nomogram)	1

TD4/4	CBSU set for Reusable Artificial Chamber	Qty
19303	CBm Turbine	1
19353	Turbine Hose	1
19120	Nitrogen Gas Supply Hose	1
19171	Reusable Guide Ring for CBm Turbine	1
22519139	Sterilization Box & Tray for CBm and One Use Turbines	1
19117	Moist Chamber	1
65078	CBm-CBSU Turbine for Artificial Chamber User Manual	1
65082	CBSU Head Selection (Nomogram)	1

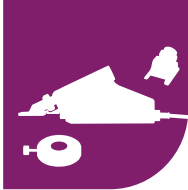SETS OF EQUIPMENT: SINGLE-USE

Sets d'équipements: Usage unique
Set of equipamientos: Desechable


GO

EO

Automated system	Manual system	
One Use-Plus Automated DSAEK TDM/3	One Use DSAEK TD1/3	CBm DSAEK TD4/3

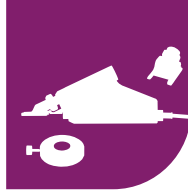SETS OF EQUIPMENT: SINGLE-USE

Sets d'équipements: Usage unique
Set of equipamientos: Desechable


G0	ACP Pack	Qty
19188	ACP System	1
19189	ACP Footswitch	1
19190	ACP Power Supply	1
19191	ACP Carrying Case	1
19450	ACP Supply Cord (EEC)	1
19451	ACP Supply Cord (USA)	1
19452	ACP Supply Cord (UK)	1
65112	ACP System User Manual	1
65114	Annex of ACP System User Manual	1

E0	Evolution 3e Control Unit	Qty
19380	Evolution 3E Console	1
19361	Evolution 3E Footswitch	1
19362	Evolution 3 Supply Cord Type CFF (CEE)	1
19363	Evolution 3 Supply Cord Type UL & CSA (USA)	1
65060/INTL	Evolution 3E User Manual	1
65073	Annex of Evolution 3E User Manual	1
19511	Shipping Carrying Case	1

TDM/3	OUP set for Disposable Artificial Chamber	Qty
19175	One Use-Plus Handpiece for Artificial Chamber	1
65106	Addendum of OUP Handpiece for Artificial Chamber	1
22519174	Storage box for One Use-Plus Handpiece	1
19117	Moist Chamber	1
65101	One Use Head Selection (Nomogram)	1

TD1/3	One Use set for Disposable Artificial Chamber	Qty
19155	One Use Turbine	1
65007	One & One Use User manual	1
19353	Turbine Hose	1
19120	Nitrogen Gas Supply Hose	1
22519139	Sterilization Box & Tray for CBm and One Use Turbines	1
19117	Moist Chamber	1
65101	One Use Head Selection (Nomogram)	1

TD4/3	CBSU set for Disposable Artificial Chamber	Qty
19303	CBm Turbine	1
19353	Turbine Flexible	1
19120	Nitrogen Gas Supply Hose	1
22519139	Sterilization Box & Tray for CBm and One Use Turbines	1
19117	Moist Chamber	1
65078	CBm-CBSU Turbine for Artificial Chamber User Manual	1
65082	CBSU Head Selection (Nomogram)	1

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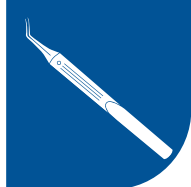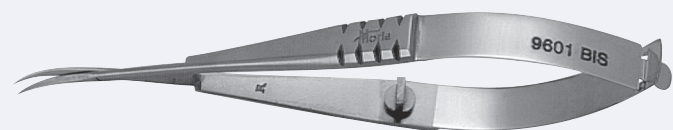
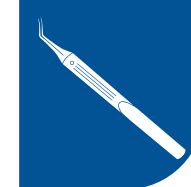
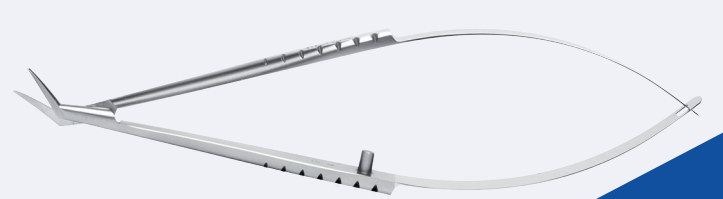
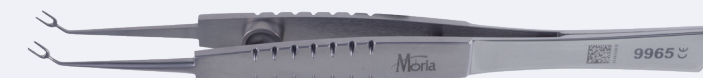
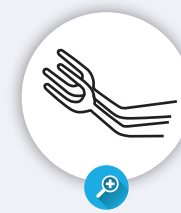
Tijeras de Katzin
Microhojas inversas, romas, 6 mm.

Remky-Polak Forceps
3 mm teeth. Positioning of the four
compass points.

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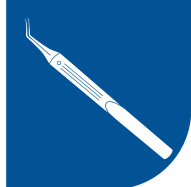
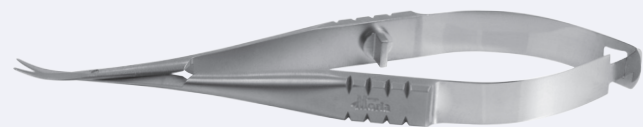
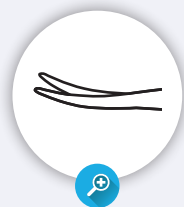
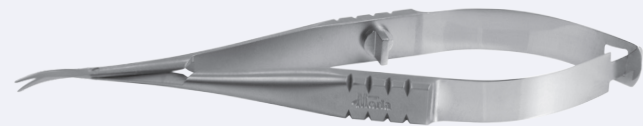
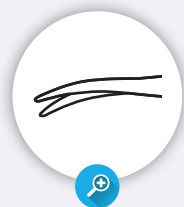
Busin Scissors

Scissors to cut the posterior stroma:

- curved
- superior blade 0.5 mm shorter than lower blade
- blunt lower blade and foam bearing surface on Descemet membrane.

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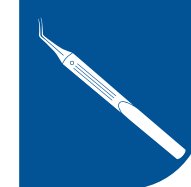
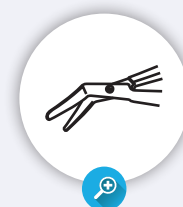
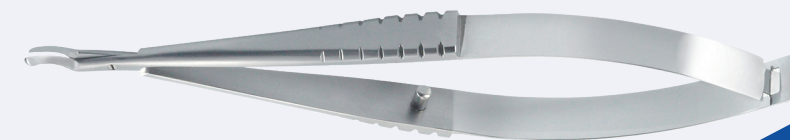
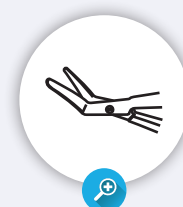
Gabison Scissors

Spatulated scissors to cut the posterior stroma:

- curved
- upper blade 0.5-mm shorter
- blunted and spatulated lower blade for safe cutting.

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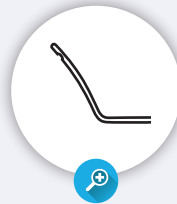
► DALK

Fontana 27G Curved Cannula

Air injection cannula to ease the "Big Bubble" creation:

- blunt and smooth tip
- curved (126° angulation) to facilitate stromal insertion
- 27G diameter shaft
- 0.4-mm diameter lower port
- air exist by posterior side.

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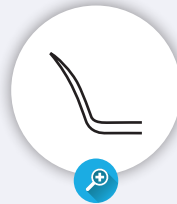
Fontana Intrastromal Dissector

Pointed dissector to create an intracorneal tunnel:

- triangle-shaped with smooth edged tip
- curved active part with a 45° angulation.

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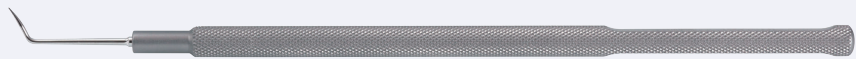
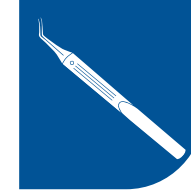
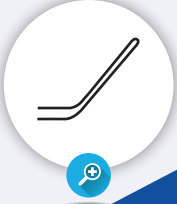
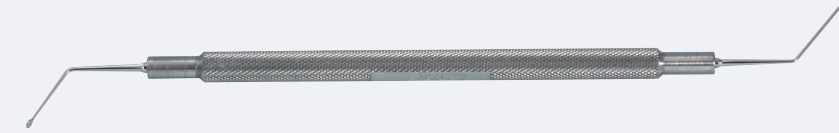
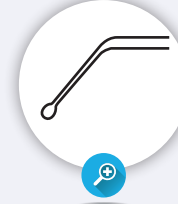
Disc-shaped spatula for detachment of intrastromal adhesions:

- semi-sharp rounded design for peripheral dissection
- blunt active part.

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Foam and flat spatula for stromal dissection:

- beveled up, blunt tip to protect Descemet's membrane while cutting posterior stroma.

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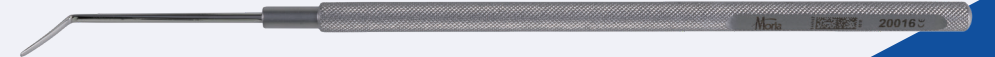
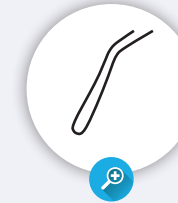
Renard Corneal Dissector Spatula Length of the tip is 10 mm.

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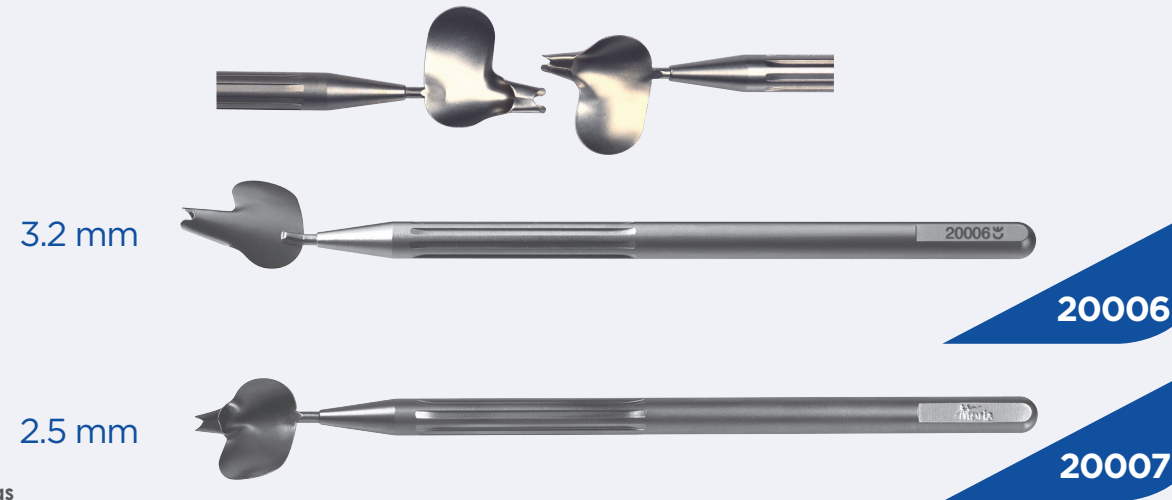
DSAEK

Busin Spatulas

For the insertion of a donor lamellar graft:

- designed for the "pull-through" technique
- lateral wing design aids in scooping up DSAEK graft directly from the Busin donor punch
- unfolding of the graft in the recipient's eye
- centration of donor button in the anterior chamber

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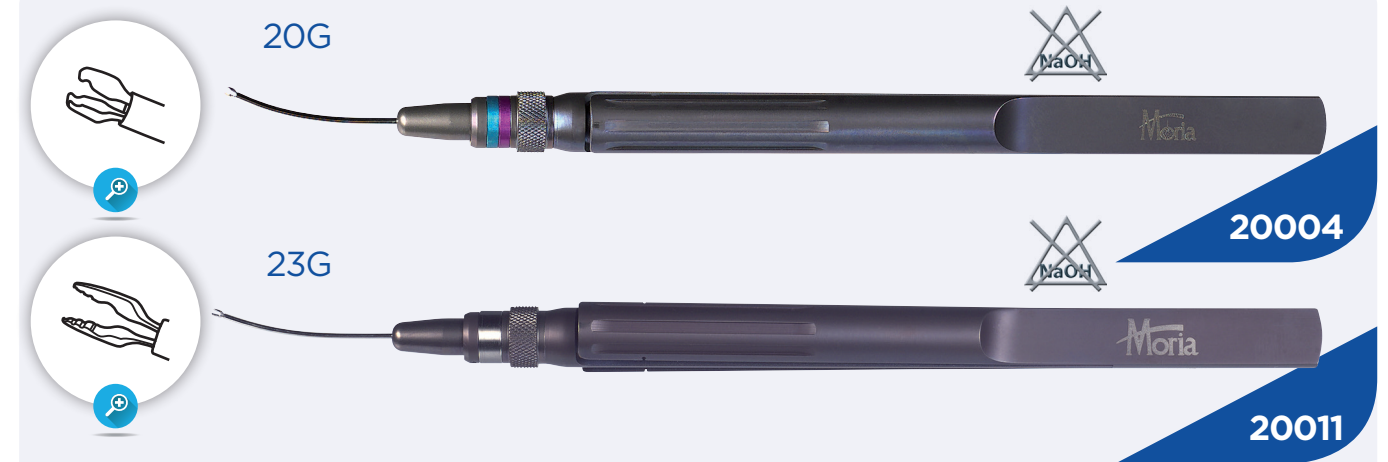
DSAEK

Busin Forceps

For grasping and positioning the donor graft:

- micro-incision forceps, distal action
- designed to position a DSAEK graft in the Busin spatula and to pull it from the spatula into the recipient's anterior chamber
- size of the tips adapted to the spatula
- atraumatic ridges to grasp the donor button.

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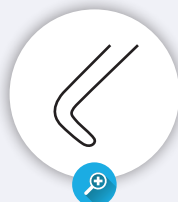
DSAEK

Price Hook
Reverse Sinskey, for Descemetorhexis.
Length of the tip is 0.7 mm.

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
19091

45° Spatula
Ideal for Descemet's stripping,
45° angled tip.

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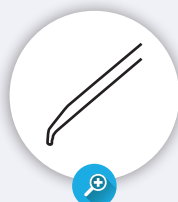Espátula 45°
Perfecta para retirar la Descemet,
punta angulada a 45°.


19077/A


Irrigating 45° Spatula
Ideal for Descemet's stripping,
45° angled tip.

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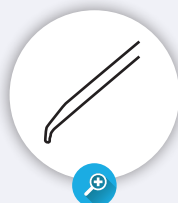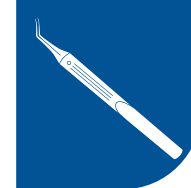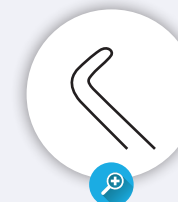
DSAEK

Hoffart Hook
Reverse Sinskey, for Descemetorhexis.
Bi-angulation with thin active part to avoid
fluid leakage at the corneal incision during
Descemetorhexis.

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
20047

90° Spatula
Ideal for Descemet's stripping,
90° angled tip.

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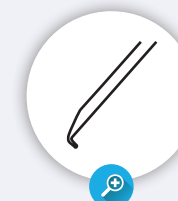Espátula 90°
Perfecta para retirar la Descemet,
punta angulada a 90°.


19077/B

Irrigating 90° Spatula
Ideal for Descemet's stripping,
90° angled tip.

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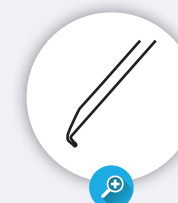DSAEK

9 mm Diameter Marker For patient cornea.

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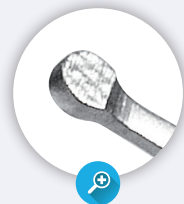
19095/850

8 mm Diameter Marker for patient cornea.

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19095/800

20003

Ogawa inserting Forceps For donor button insertion:

- blunt tips for the release of donor tissue inside the patient's eye
- angulation of handle optimized for a fingertip control
- when the instrument is closed, active tips remain slightly open not to compress the donor tissue

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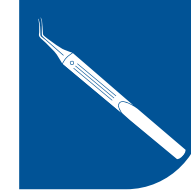
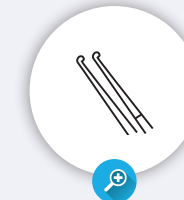
DSAEK

DSO Gorovoy Forceps Inverted Utrata shaped, blunt tip, 11.5 cm long.

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
19097

DSAEK "S" Marker Designed to mark an "S" on the stromal side of the donor button.

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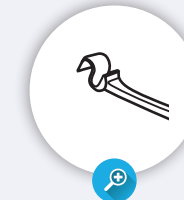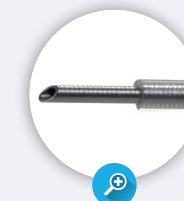
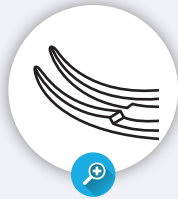
► DMEK

Curved DMEK Forceps
7° curvature, 10 mm platforms,
12 cm long.

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Pinza DMEK curva
Curvature 7°, placas de 10 mm,
longitud 12 cm.


20038

Straight DMEK Forceps
10 mm platforms, 12 cm long.

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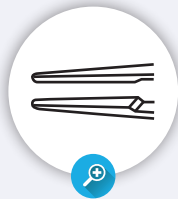20039

DSO Gorovoy Forceps
Inverted Utrata shaped,
blunt tip, 11.5 cm long.

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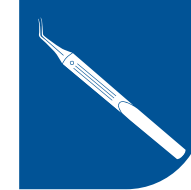
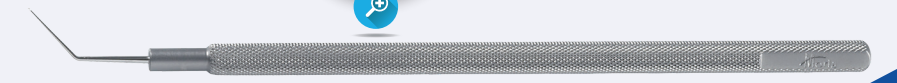
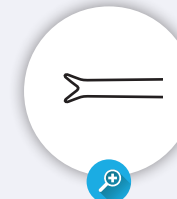
20021

Micro Dissector
For endothelial graft,
tips of 3 mm.

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► DSO

DSO Gorovoy Marker 4 mm For patient cornea

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
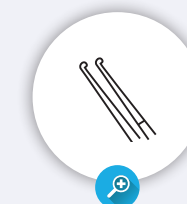
19097

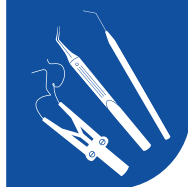PENETRATING KERATOPLASTY

BASIC SET #442A
EXTENDED SET #442B
(BASIC SET + EXTENSION)

► BASIC SET

PK

► EXTENSION

► BASIC SET

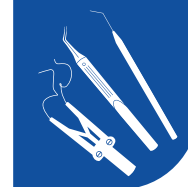
- 1. 20035** Adjustable Speculum (with curved blades)
- 2. 7252BIS** Barraquer-De Laage Needle Holder
- 3. 13161** Bonn-Moria Forceps
- 4. 8023A** Katzin Scissors (clockwise)

- 5. 8023B** Katzin Scissors (anti-clockwise)
- 6. 9601BIS** Vannas-Moria Scissors
- 7. 17120** SFO Cannula
- 8. 11051** Castroviejo-Grancini Caliper
- 9. 22518357** Stainless Sterilization Box (Not shown)

► EXTENSION

- 10. 9965** Remky-Polak fixation Forceps
- 11. 11082** Blade (pack of 10) sterile
- 12. 8065** Troutman Blade Holder
- 13. 17150B** Silicone Block Receptacle Hollowed

Moria


DALK


BASIC SET #441A
EXTENDED SET #441B
(BASIC SET + EXTENSION)

► BASIC SET

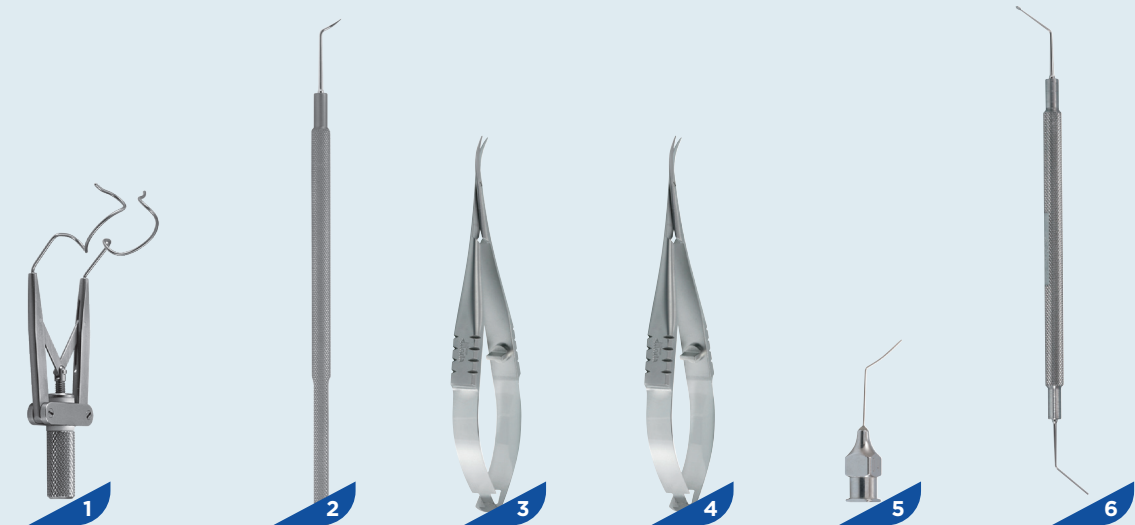DALK

► EXTENSION

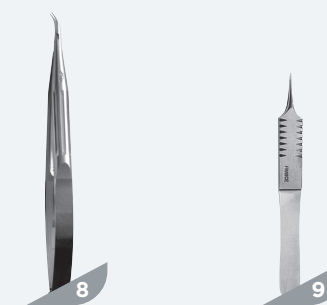► BASIC SET

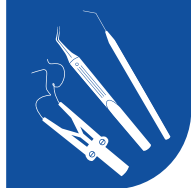
- 1. 20035** Adjustable Speculum (with curved blades)
- 2. 20026** Fontana Intrastromal Dissector
- 3. 20024** Busin Scissors (clockwise)
- 4. 20025** Busin Scissors (counter-clockwise)

- 5. 20027** Fontana 27G Curved Cannula
- 6. 20028** Fontana Double-Ended Spatula
- 7. 22518358** Stainless Sterilization Box (Not shown)

► EXTENSION

- 8. 7252BIS** Barraquer-De Laage Needle Holder
- 9. 7850A** Bonn-Moria Forceps

Moria


DSAEEK


BASIC SET #439A
EXTENDED SET #439B
 (BASIC SET + EXTENSION)

► BASIC SET

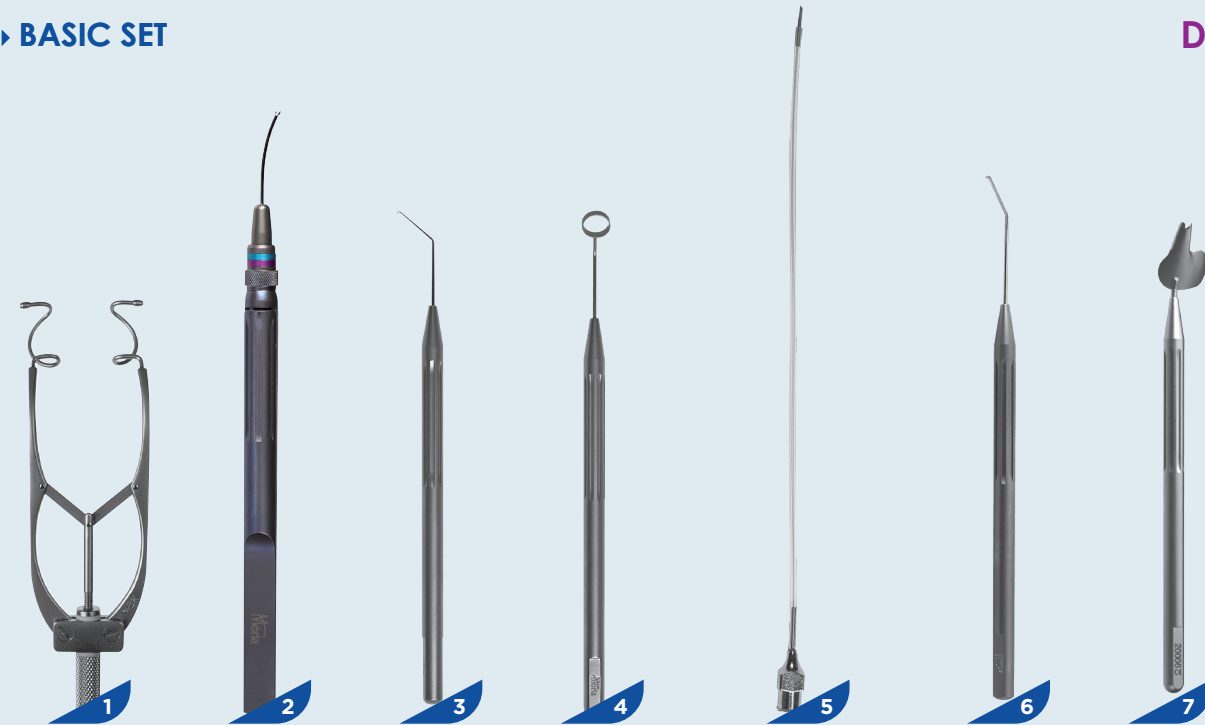DSAEEK

► EXTENSION

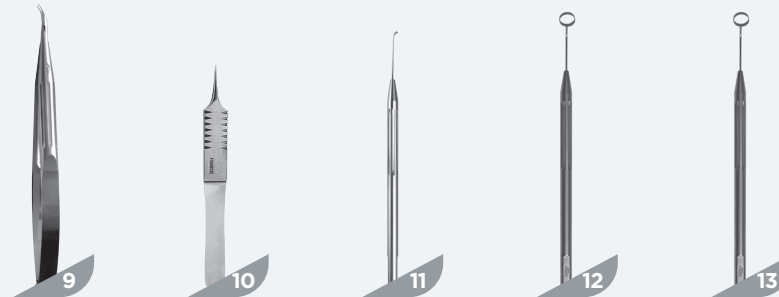
► BASIC SET

1. **18195** Schapira Speculum
2. **20004** Busin Forceps
3. **19091** Price Endothelial Keratoplasty Hook
4. **19095** 9 mm Marker

5. **19092** Chamber Maintainer 20G
6. **19077/B** 90° Spatula
7. **20006** Busin Spatula
8. **22519093** Aluminium Sterilization Box (Not shown)

► EXTENSION

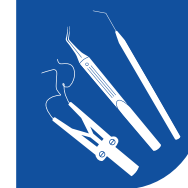9. **7252BIS** Barraquer-De Laage Needle Holder
10. **7850A** Bonn-Moria Forceps
11. **19096** DSAEK "S" Marker
12. **19095/850** 8.5 mm Marker
13. **19095/800** 8 mm Marker


Ultra-Thin DSAEK


BASIC SET #440A
EXTENDED SET #440B
 (BASIC SET + EXTENSION)

► BASIC SET

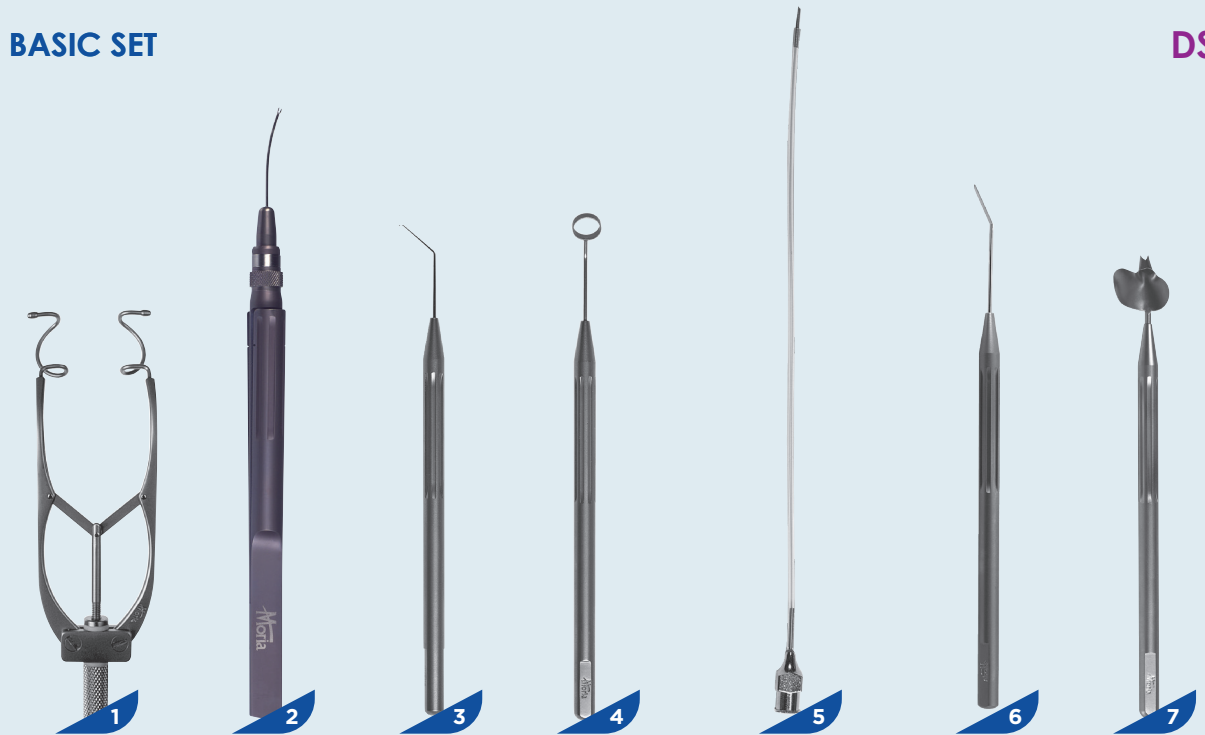DSAEEK

► EXTENSION

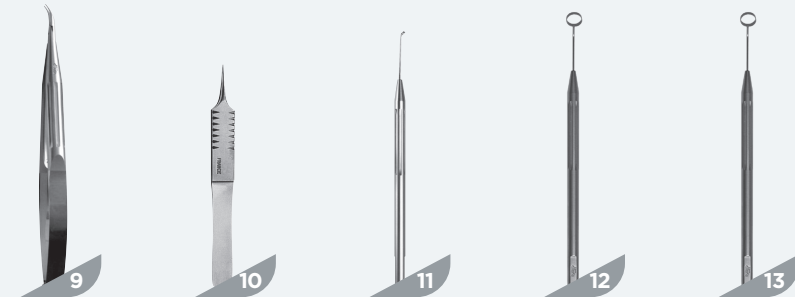
► BASIC SET

1. **18195** Schapira Speculum
2. **20011** Busin Forceps 23G
3. **19091** Price Endothelial Keratoplasty Hook
4. **19095** 9 mm Marker
5. **19092** Chamber Maintainer 20G

6. **19077/B** 90° Spatula
7. **20007** Busin Spatula
8. **22519093** Aluminium Sterilization Box (Not shown)

► EXTENSION

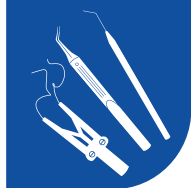9. **7252BIS** Barraquer-De Laage Needle Holder
10. **7850A** Bonn-Moria Forceps
11. **19096** DSAEK "S" Marker
12. **19095/850** 8.5 mm Marker
13. **19095/800** 8 mm Marker


DMEK

Graft preparation

BASIC SET #438A


► BASIC SET

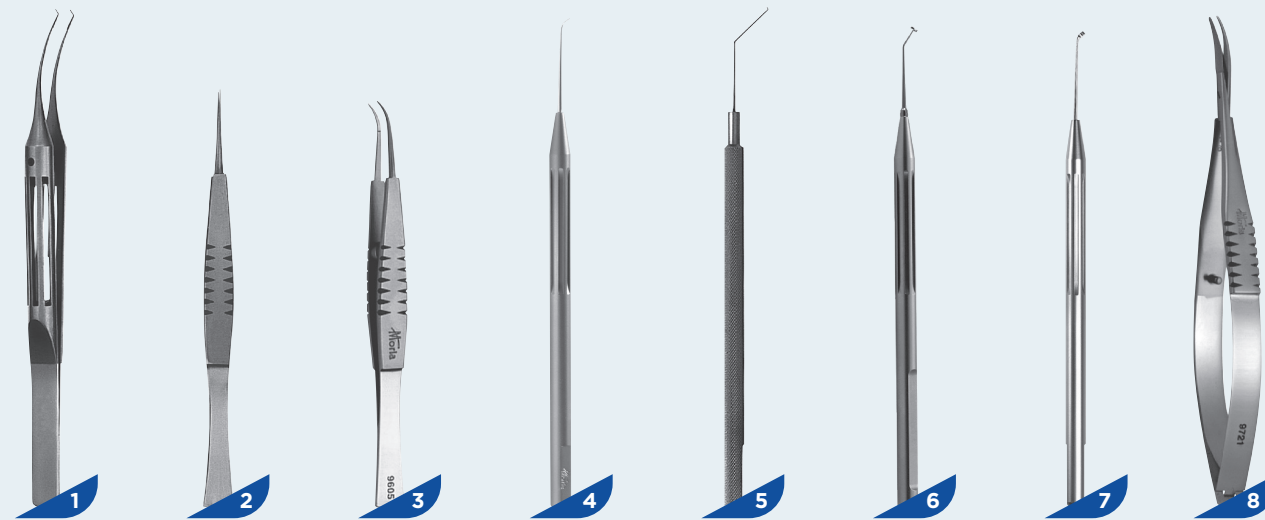DMEK

► BASIC SET

- | | | | | | |
|----------|-----------------------------|----------|-----------------------------|-------------|---|
| 1. 13240 | Kraff-Moria Forceps | 4. 20022 | Micro Dissector | 7. 19096 | DSAEK "S" Marker |
| 2. 9604 | Troutman Forceps (straight) | 5. 13225 | Buratto-Sinskey Manipulator | 8. 9721 | Castroviejo Scissors |
| 3. 9605 | Troutman Forceps (curved) | 6. 20034 | Angled «S» Marker | 9. 22518353 | Aluminium Sterilization Box (Not shown) |

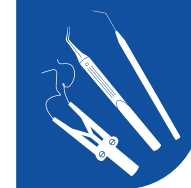
DMEK

Surgery

BASIC SET #437A

► BASIC SET

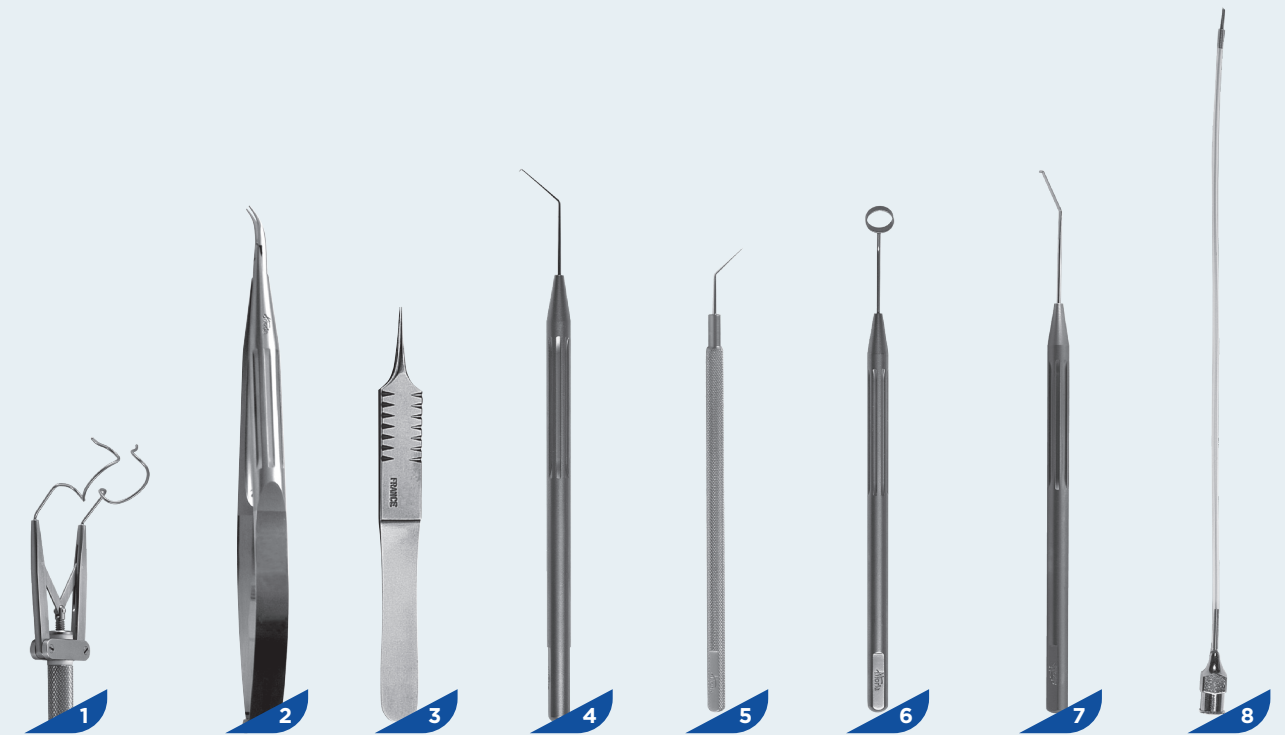DMEK

► BASIC SET

- | | | | | | |
|------------|--|------------|-------------------------------------|-------------|---|
| 1. 20035 | Adjustable Speculum (with curved blades) | 4. 19091 | Price Endothelial Keratoplasty Hook | 8. 19092 | Chamber Maintainer 20G |
| 2. 7252BIS | Barraquer-De Laage Needle Holder | 5. 20021 | Donor Descemet's Stroma Scorer | 9. 22518353 | Aluminium Sterilization Box (Not shown) |
| 3. 7850A | Bonn-Moria Forceps | 6. 19095 | 9 mm Marker | | |
| | | 7. 19077/B | 90° Spatula | | |

SINGLE-USE INSTRUMENTS

Instruments à usage unique / Instrumentos desechables

DSAEK

Marker for Patient Cornea
Available in 8-, 8.5- and 9-mm diameter
Box of 10 units

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9 mm

17520X10

Price Hook
Reverse Sinskey, recommended in DSAEK
and DMEK for the Descemetorhexis
Box of 5 units

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17302X5

90° Spatula
90° angled tip, recommended in DSAEK and
DMEK for Descemet's stripping
Box of 5 units

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Instruments à usage unique / Instrumentos desechables

DSAEK

Busin Spatula
Recommended in DSAEK
For insertion of donor graft; 3.2 mm incision
Box of 5 units

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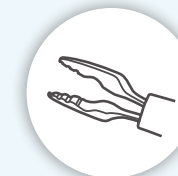
17300X5

Busin Forceps 23G
Recommended in DSAEK for grasping
and positioning the donor graft
Box of 1 or 5 units

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17301X1

17301X5

Chamber Maintainer 23G
French distribution only.

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9 mm

17520X10

Price Hook
Reverse Sinskey, recommended in DSAEK and DMEK for the Descemetorhexis
Box of 5 units

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17302X5

90° Spatula
90° angled tip, recommended in DSAEK and DMEK for Descemet's stripping
Box of 5 units

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Instruments à usage unique / Instrumentos desechables

► DMEK

Spatula for the Corneal Epithelium
Box of 10 units

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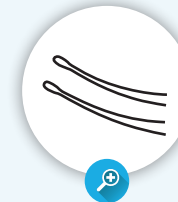
17541X10

Graft Preparation Forceps
1.2 mm oval-shaped tip
Curved, 15 mm long
Box of 10 units

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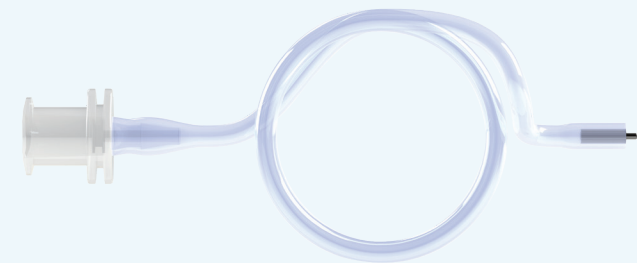
<https://www.moria-surgical.com/product/single-use-chamber-maintener>

M2658

OVERVIEW OF EQUIPMENT REQUIRED PER CORNEAL SURGERY

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OVERVIEW OF INSTRUMENTS REQUIRED PER CORNEAL SURGERY

Aperçu des instruments requis selon la procédure chirurgicale
Visión de los instrumentos per queratoplastia

		PK	DALK	ALTK	DSAEK	DMEK
Reusable	Donor	#442A #442B	#441A #441B	#439A #439B	#440A #440B	#438A
	Recipient					#437A
Single-Use	Donor			#17300X5 #17301X1 - #17301X5 #17541X10		#17521X10
	Recipient			#17518X10 #17519X10 #17520X10 #17302X5 #17303X5 #17303X10 #17541X10		#17518X10 #17519X10 #17520X10 #17302X5 #17303X5 #17303X10 #17541X10

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17214DXXX **Deep Well Full-Cut Punch**
Punch Deep Well Full-Cut
Punch Deep Well Full-Cut16

17215RXXX **Deep Well Tri-Fold Cover**
Porte-lame Profond Tri-Fold
*Soporte de cuchilla Profundido Tri-Fold....*20

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20003 **Ogawa Inserting Forceps**
Pince d'Ogawa
Pinza de Ogawa40

20004 **Busin Forceps 20G**
Pince de Busin 20G
Pinza de Busin 20G37, 50

20006 **Busin Spatula**
Spatule glissière de Busin
Espátula corredora de Busin36, 50

20007 **Busin Spatula**
Spatule glissière de Busin
Espátula corredora de Busin36, 51

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ALPHABETICAL INDEX

A	ACP Pack G0	8
	Adjustable Trephine 17202DXXX	22
	ALTK Applanating Lenses for Donor 19165	24
	ALTK Applanating Lenses for Donor 19166	24
	ALTK Applanating Lenses for Recipient 19310	24
	ALTK Applanating Lenses for Recipient 19311	24
	Angled « S » Marker 20034	46, 52
	Aspiration tubing 19138	9
B	Busin Forceps 23G 17301XX	55
	Busin Forceps 20G 20004	37, 50
	Busin Forceps 23G 20011	37, 51
	Busin Punch 17200DXXX	16
	Busin Scissors (Anti-clockwise) 20025	32, 49
	Busin Scissors (Clockwise) 20024	32, 49
	Busin Spatula 17300X5	55
	Busin Spatula 20006	36, 50
	Busin Spatula 20007	36, 51
C	CBm Turbine 19303	10
	CBm-ALTK suction ring for recipient 19309/X	13
	CBSU set for Reusable Artificial Chamber TD4/4	26
	Chamber Maintainer 20G 19092	41, 43, 50, 51, 53
	Chamber Maintainer 23G M2658	55, 57
	Curved DMEK Forceps 20038	42
D	Deep Well Full-Cut Punch 17214DXXX	16
	Deep Well Guarded Cover 17213DXXX	18
	Deep Well Guarded Punch 17212DXXX	18
	Deep Well Tri-Fold Cover 17215RXXX	20
	Donor Descemet's Stroma Scorer 20021	43, 53
	DSAEK "S" Marker 19096	41, 50, 51, 52
	DSO Gorovoy Forceps 19097	41, 42, 47
	DSO Gorovoy Marker 4 mm 20043	47
E	Evolution 3E console to operate One Use-Plus, One Use and CBm systems E0	9
F	Fontana 27G Curved Cannula 20027	34, 49
	Fontana Double-Ended Spatula 20028	35, 49
	Fontana Intrastromal Dissector 20026	34, 49
G	Gabison Scissors (Anti-clockwise) 20030	33
	Gabison Scissors (Clockwise) 20029	33
	Graft Preparation Forceps 17521X10	57
	Guarded Punch 17207DXXX	19
H	Handle for Dezard Trephine 13330	23
	Hoffart Hook 20047	39, 45

I	Irrigating 45° Spatula 19083/A	38, 44
	Irrigating 90° Spatula 19083/B	39, 45
K	Katzin Scissors (anti-clockwise) 8023B	31, 48
	Katzin Scissors (clockwise) 8023A	31, 48
M	Marker for Patient Cornea 8 mm 17518X10	54, 56
	Marker for Patient Cornea 8.5 mm 17519X10	54, 56
	Marker for Patient Cornea 9 mm 17520X10	54, 56
	Marker for Patient Cornea 8 mm 19095/800	40, 46, 50, 51
	Marker for Patient Cornea 8.5 mm 19095/850	40, 46, 50, 51
	Marker for Patient Cornea 9 mm 19095 ...	40, 46, 50, 51, 53
	Micro Dissector 20022	43, 52
	Moist Chamber 19117	19
	Muraine Punch 17209D800	20

O	Ogawa inserting Forceps 20003	40
	One Use set for Disposable Artificial Chamber TD1/3	28
	One Use set for Disposable Artificial Chamber TD4/3	26, 28
	One Use set for Reusable Artificial Chamber TD1/4	26
	One Use Turbine 19155	10
	One Use-Plus Handpiece 19175	10
	OUP set for Disposable Artificial Chamber TDM/3	28
	OUP set for Reusable Artificial Chamber TDM/4	26

P	Price Hook 19091	38, 44, 50, 51, 53
	Price Hook 17302X5	54, 56
R	Remky-Polak Forceps 9965	31, 48
	Renard Corneal Dissector Spatula 20016	35
	Reusable Artificial Chamber-Base 19161	15
	Reusable Artificial Chamber - Base & Cover ALTKD/2	15, 26
	Reusable Artificial Chamber-CBm cover 19172	15
	Reusable Artificial Chamber-Cover 19162	15
	Reusable Dezard trephine with limit stop 18186/15	23
	Reusable Donor Punch 17169	17
	Reusable Guide-Ring for CBm turbine 19171	13
	Reusable Guide-Ring for One Use turbine 19168	12
	Reusable Guide-Ring for One Use-Plus handpiece 19173 ..	12
	Reusable Heads for CBm turbine 19170/XXX	11
	Reusable Silicone Block Receptacle Hollowed 17150B	23, 48
	Reusable Trephine Blade 17150DXX	23

S	SET DALK Basic set #441A	49
	SET DALK Extended set #441B	49
	SET DMEK Graft preparation Basic set #438A	52
	SET DMEK Surgery Basic set #437A	53
	SET DSAEK Basic set #439A	50
	SET DSAEK Extended set #439B	50
	SET Penetrating Keratoplasty Basic set #442A	48
	SET Penetrating Keratoplasty Extended set #442B	48
	SET Ultra-Thin DSAEK Basic set #440A	51
	SET Ultra-Thin DSAEK Extended set #440B	51
	Simple Trephine 17201DXXX	22
	Single-Use Artificial Chamber for Single-Use Trephines 17204	14
	Single-Use Artificial Chamber 19182	14
	Single-Use Guide-Ring for CBm turbine 19180	13
	Single-Use Guide-Ring for One Use turbine 19186	12
	Single-Use Guide-Ring for One Use-Plus handpiece 19187	12
	Single-Use Heads for CBm turbine 19178/xxx	11
	Single-Use Heads for One Use-Plus and One Use 19184/XXX	11
	Single-Use Tubing Set for ACP 19192	8
	Single-Use Tubing Set for Artificial Chamber 19181	15
	Spatula for the Corneal Epithelium 17541X10	57
	Spatula 45° 19077/A	38, 44
	Spatula 90° 17303XX	54, 56
	Sterile Blades for Donor Punch 17171DXXX2	17
	Sterilisation tray for CBm heads & rings 22519343	25
	Sterilization box & tray for CBm and One Use turbines 22519139	25
	Sterilization box for reusable artificial 22519167	25
	Storage box for ALTK applanating lenses 22519169	25
	Storage box for One Use-Plus microkeratome for artificial chamber 22519174 s	25
	Straight DMEK Forceps 20039	42

T	Teflon® Receptacle Block for Donor Punch 17171P	17
	Troutman Scissors (anti-clockwise) 9851)	30
	Troutman Scissors (clockwise) 9850	30
	Two-colored Teflon®/Propylux® Receptacle Block for Donor Punch 17171PB	17
V	Vannas-Moria Scissors 9601BIS	30, 48
X	XL Guarded Cover 17206DXXX	19
	XL Guarded Punch 17208DXXX	19

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Set DSAEK Basic set **#439A** 50

Set DSAEK Extended set **#439B** 50

Set Penetrating Keratoplasty Basic set **#442A** 48

Set Penetrating Keratoplasty Extended set **#442B** 48

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HALK: with CBm-ALTK system & Reusable Hanna Donor Punch

- **Tan et al.** Hemi-automated lamellar keratoplasty (HALK). *British J Ophthalmol.* **2011**;95(11):1513-1518
- **Mehra et al.** Long-term outcomes of HALK. *Clin Exp Ophthalmol.* **2018**;46(9):1017-1027

DALK: with Single-Use Vacuum-Assisted Adjustable Recipient Trephine

- **Moshirfar et al.** Accuracy of corneal trephination depth using the Moria single-use adjustable depth vacuum trephine system. *Clin Ophthalmol.* **2014**;8:2391-2396
- **Busin et al.** Outcomes of air injection within 2 mm inside a deep trephination for DALK in eyes with keratoconus. *American J Ophthalmol.* **2016**;164(4):6-13
- **Busin et al.** Deep trephination allows high rates of successful pneumatic dissection for DALK independent of surgical experience. *Cornea* **2019**;38(5):645-647

DSAEK: With One Use manually-driven turbine:

- **Busin et al.** Postoperative graft thickness obtained with single-pass microkeratome-assisted UT DSAEK. *Cornea* **2015**;34(11):1362-1364
- **Touboul et al.** Reproducibility of single-pass donor DSAEK tissue preparation with the Moria single-use microkeratome. *French J Ophthalmol.* **2016**;39(9):780-785
- **Borderie et al.** Influence of graft thickness and regularity on vision recovery after endothelial keratoplasty. *British J Ophthalmol.* **2020**;104(9):1317-1323
- **Busin et al.** UT-DSAEK with the microkeratome double-pass technique: two-year outcomes. *Ophthalmology* **2013**;120(6):1186-1194

DMEK: With Reusable Vacuum-Assisted Hanna Donor Punch & Evolution 3E Console:

- **Melles et al.** Descemet Membrane Endothelial Keratoplasty (DMEK). *Cornea* **2006**;25(8):987-990
- **Kruse et al.** A stepwise approach to donor preparation and insertion increases safety and outcome of DMEK. *Cornea* **2011**;30(5):580-587
- **Kruse et al.** Characterization of the cleavage plane in DMEK. *Ophthalmology* **2011**;118(10):1950-1957
- **Kruse et al.** Reproducibility of graft preparations in DMEK. *Ophthalmology* **2013**;120(9):1769-1777
- **Kruse et al.** Long-term clinical outcome after DMEK. *American J Ophthalmol.* **2016**;169:218-226
- **Seitz et al.** Anatomy-based DMEK wetlab in Homburg/Saar: novel aspects of donor preparation and host maneuvers to teach DMEK. *Clin Anatomy.* **2018**;31(1):16-27
- **Seitz et al.** DMEK - Spender und empfänger schritt für schritt. *Ophthalmologie* **2020**;117(8):811-828
- **Seitz et al.** Reproducibility of non-invasive endothelial cell loss assessment of the prestripped DMEK roll after preparation and storage. *American J Ophthalmol.* **2021**;221(1):17-26

With Single-Use Vacuum-Assisted Guarded Donor Punch:

- **Eghrari et al.** Viability of DMEK grafts folded in the eye bank. *Cornea* **2018**;37(11):1474-1477

With Single-Use Vacuum-Assisted Muraine Guarded Donor Punch:

- **Muraine et al.** Novel technique for the preparation of corneal grafts for DMEK. *American J Ophthalmol.* **2013**;156(5):851-859

DSAEK: With One Use-Plus automated microkeratome & ACP system:

- **Shimazaki et al.** Comparison of artificial anterior chamber internal pressures and cutting systems for DSAEK. *Trans Vis Sci Tech.* **2018**;7(6):11
- **Quaranta et al.** Single-pass microkeratome and ACP for the Ultra-Thin DSAEK graft preparation. *Cornea* **2021**;40(6):755-763

Legal notices for healthcare professionals

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Carefully read the instructions in the package leaflet or on the label given to the healthcare professional.

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MORIA Inc

1050 Cross Keys Drive
Doylestown, PA 18902
USA
Phone: (800) 441 1314
Fax: +1 (215) 230 7670
www.moria-surgical.com

MORIA Japan K.K.

Arcadia Building 6F
1-12-3 Kanda
SudachoChiyoda-Ku
Tokyo 101-0041
JAPAN
Phone: 81-3-6260-8309
Fax: 81-3-6260-8310
www.moriajapan.com

**MORIA Commercial
(CHINA) CO., LTD.**

上海目利亚贸易有限公司
Room H 6F Kaili Building NO.432
West Huai Hai Road, Changning
district, Shanghai, 200052, P.R.C
中国上海市长宁区淮海西路432号凯
利大厦6楼H室
Tel/Fax: +86 021 52586095
www.moria-surgical.com