

3!FLEX

PRELOADED



3 CLICKS FOR EASY,
SAFE AND PREDICTABLE
INJECTION

MEDI  **NTUR**

Vision of Expertise


MEDJET PL-MA 2.2

Single use injection system for Bi-Flex preloaded monofocal, toric and diffractive progressive hydrophilic IOLs



1st CLICK LOADING

Insert the injector to the wet container

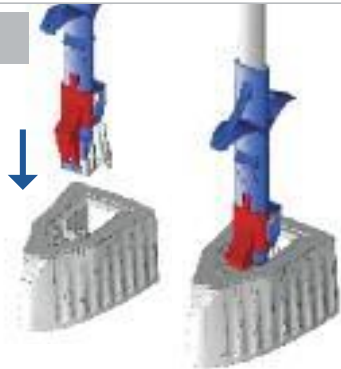


2nd CLICK CLOSING

Fold up the cartridge nozzle



3rd CLICK READY

Push the red stopper forward to the end position and fill visco

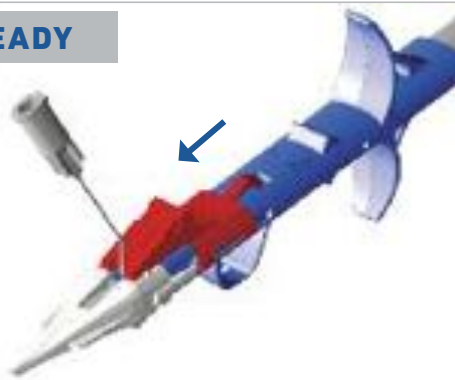

INJECTION

Push the plunger slowly to the end position



Lens types	Single-piece aspheric IOLs, clear and yellow <ul style="list-style-type: none"> • monofocal • toric • diffractive progressive (Patent pending)
-------------------	---

Diameters 6.0 mm (optic) / 13.0 mm (overall)

Angulation 0° – asymmetrical design

Material Copolymer of hydrophobic and hydrophilic monomers

Incision size 2.2 mm

PCO protection 360° Special Square Edge

A-Constant 118.9



677P/677PY 25% water content with UV Absorber + blue light filter (390–470 nm)
 Powers available -10.0 D → +9.0 D · (1.0 D steps)
 +10.0 D → +30.0 D · (0.5 D steps)



677TP/677TPY 25% water content with UV Absorber + blue light filter (390–470 nm)
 Powers available -10.0 D → +30.0 D · (0.5 D steps)
 Cylinders available 1.00 D, 1.50 D, 2.25 D, 3.00 D, 3.75 D, 4.50 D



677MPY 25% water content with UV Absorber + blue light filter (390–470 nm)
 Powers available 0.0 D → +30.0 D (0.5 D steps)
 Addition +3.5 D

LA-IMCMDJPLMA 02 201409