

Q FLEX

PRELOADED




3 CLICKS FOR EASY,
SAFE AND PREDICTABLE
INJECTION

MEDI  **NTUR**
Vision of Expertise


MEDJET PIL-MA

Single use injection system for Q-Flex preloaded monofocal and diffractive progressive hydrophilic IOLs

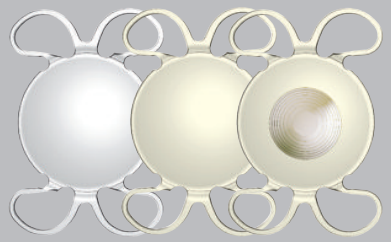


1st CLICK LOADING

Insert the injector to the wet container

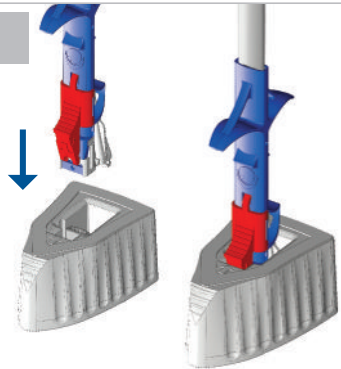


2nd CLICK CLOSING

Fold up the cartridge nozzle

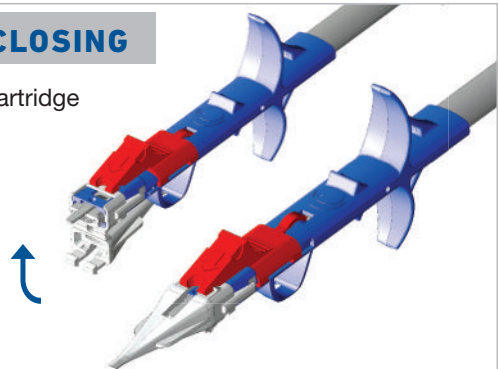


3rd CLICK READY

Push the red stopper forward to the end position and fill visco

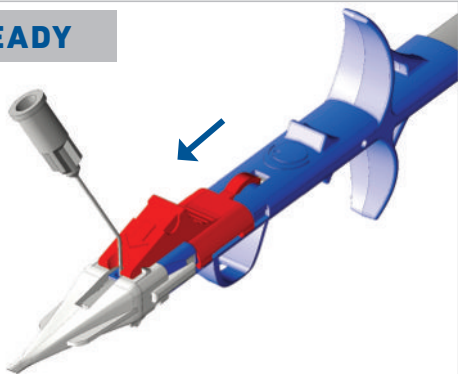

INJECTION

Push the plunger slowly to the end position

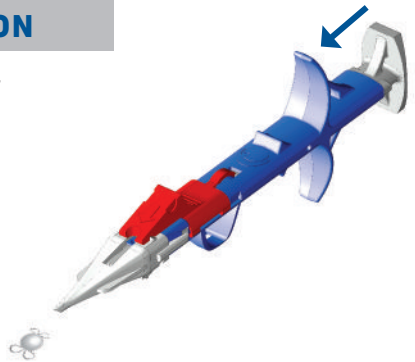

Lens types Single-piece aspheric IOLs, clear and yellow
 • monofocal
 • diffractive progressive
 (Patent pending)

Optic diameter 6.0 mm

Overall diameters
 0.0 D → +15.0 D · 11.0 mm
 +15.5 D → +22.0 D · 10.7 mm
 +22.5 D → +30.0 D · 10.5 mm


Angulation 0°

Material Copolymer of hydrophilic and hydrophobic monomers


Incision size 2.2 mm

PCO protection 360° Special Square Edge

A-Constant 118.9 (SRK/T)



640P/640PY
 25% water content with UV Absorber / + blue light filter (390–470 nm)
 Powers available
 0.0 D → +30.0 D · (0.5 D steps)



640PMY 25% water content with UV Absorber / + blue light filter (390–470 nm)
 Powers available
 0.0 D → +30.0 D · (0.5 D steps)
 Addition 3.0 D; +3.5 D

LA-IMCOPLMAENG 01 201505



Medicontur
 Medical Engineering Ltd
 export@medicontur.com
 www.medicontur.com

Head Office
 Herceghalmi Road
 2072 Zsámbék
 Hungary

International Office
 Chemin des Aulx 18
 1228 Plan-les-Ouates
 Switzerland / Geneva

